

PAY ATTENTION!

Important instructions for your Philips FRx defibrillator.

Activate your defibrillator immediately.

- Pull the green plastic strip out of the defibrillator
- The defibrillator will now perform a self-test
- Do not press the green button
- Press the orange button when indicated
- Press the green button when indicated (press it now)
- The self-test of the defibrillator takes about 1 minute (during the self-test, the defibrillator regularly says 'test')
- If the defibrillator says 'ready for use', the self-test is completed
- Check that the green light (top right) is flashing. This light flashes green every 4 seconds, showing that the defibrillator is ready for use
- Place the defibrillator in the carrying case and close it. You can see the green light through the sight glass.
- If present:
 - Quick intervention kit: attach it to the bag with the hook provided
- FRx baby/child key: Open the carrying case. On the left side is a flap that you can open with Velcro. Open this flap and place the baby/child key in the slide-in pocket provided for this purpose