

PAY ATTENTION!

Important instructions for your Philips HS1 defibrillator. Activate your defibrillator immediately.

1. Pull the green plastic strip out of the defibrillator
2. The defibrillator will now performs a self-test
3. Do not press the green button
4. Press the orange button when it flashes
5. The self-test of the defibrillator takes about 1 minute (during the self-test, the defibrillator regularly says 'test')
6. If the defibrillator says 'ready for use', the self-test is completed
7. Check that the green light (top right) is flashing. This light flashes green every 4 seconds, showing that the defibrillator is ready for use
8. Place the defibrillator in the carrying case and close it. You'll see the green light through the viewing window
9. Attach the Quick Intervention Kit to the bag with the included hook